



## Microsoft® Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES4K, nCare™ MX/ RX, EasyLink™ Router, LiveStream™ StatusBoard Engine, and VaultStream™ systems.

Also included is information on the latest PHP updates approved for the v10.x and up VaultStream™ systems.

**Products:** EasySuite™ HD, ES4K, and nCares™, connected to a network and VaultStream™, as well as the LiveStream™ StatusBoard Engine.

**Date:** December 14, 2021

### **Critical Security Bulletins (Obtained from Microsoft):**

**NOTE:** Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813, or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

### **Approved Updates:**

| <b>December 2021 Updates</b>                             |                  |  |
|--|------------------|--|
| <b>KB Number</b>   | <b>Platform</b>  | <b>Title</b>   |
| <b>VaultStream™ v10.x Updates – Windows® Server 2012</b> |                  |  |
| KB5008263  | VaultStream 2012 | 2021-12 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems |
| KB890830   | VaultStream 2012 | Windows Malicious Software Removal Tool x64 - v5.96                                      |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.274.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.242.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.208.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.194.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.156.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.150.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.128.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.125.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.104.0)   |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.73.0)    |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.38.0)    |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2306.0)  |
| KB2310138  | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2293.0)  |

|           |                  |   |
|-----------|------------------|---|
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2273.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2250.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2226.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2204.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2167.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2153.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2124.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2119.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2106.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2078.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2038.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.2028.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1986.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1971.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1952.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1914.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1908.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1825.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1793.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1762.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1751.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1721.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1697.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1684.0) |

|           |                  |   |
|-----------|------------------|---|
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1644.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1638.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1553.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1542.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1528.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1502.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1477.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1449.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1418.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1394.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1379.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1365.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1342.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1337.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1293.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1279.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1236.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1219.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1196.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1163.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1139.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1059.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1057.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.353.1035.0) |
|           |                  |   |

| <b>VaultStream™ v10.x Updates – Windows® Server 2016</b>  |                          |   |
|---|--------------------------|---|
| KB5008207   | VaultStream 2016         | 2021-12 Cumulative Update for Windows Server 2016 for x64-based Systems                         |
|   |                          |   |
| <b>VaultStream™ v10.4 Updates – Windows® Server 2019</b>  |                          |   |
| KB5007406   | VaultStream 2019         | 2021-11 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 |
| KB5008218   | VaultStream 2019         | 2021-12 Cumulative Update for Windows Server 2019 (1809) for x64 based Systems                  |
| KB2267602   | VaultStream 2019         | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.292.0)           |
| KB2267602   | VaultStream 2019         | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.305.0)           |
| KB2267602   | VaultStream 2019         | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.316.0)           |
| KB2267602   | VaultStream 2019         | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.346.0)           |
| KB2267602   | VaultStream 2019         | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.359.0)           |
| KB4052623   | VaultStream 2019         | Update for Microsoft Defender Antivirus antimalware platform - (Version 4.18.2111.5)            |
| KB890830  | VaultStream 2019         | Windows Malicious Software Removal Tool x64 - v5.96   |
|   |                          |   |
| <b>nCare™ MX v10.x</b>                                    |                          |   |
| KB5008207   | nCare MX                 | 2021-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems                     |
|   |                          |   |
| <b>ES4K v10.x   ES4K Controller (a.k.a. BlizCon)</b>      |                          |   |
| KB5008207   | ES4K Controller          | 2021-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems                     |
|   |                          |   |
| <b>ES4K v10.x   ES4K Video Manager (a.k.a. nCare RX2)</b> |                          |   |
| KB5008207   | ES4K Video Manager       | 2021-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems                     |
|   |                          |   |
| <b>ES4K v10.x – LiveStream™ StatusBoard 2</b>             |                          |   |
| KB5008218   | LiveStream StatusBoard 2 | 2021-12 Cumulative Update for Windows 10 Version 1809 for x64-based Systems                     |
| KB4052623   | LiveStream StatusBoard 2 | Update for Microsoft Defender Antivirus antimalware platform - (Version 4.18.2111.5)            |
| KB2267602   | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.324.0)           |
| KB2267602   | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.273.0)           |
| KB2267602   | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.210.0)           |
| KB2267602   | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.161.0)           |



|           |                             |   |
|-----------|-----------------------------|---|
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.355.119.0)  |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.355.70.0)   |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.355.9.0)    |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.2300.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.2250.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.2195.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.2148.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.2096.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.2047.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1986.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1934.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1888.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1825.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1762.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1712.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1653.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1607.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1546.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1502.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1438.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1387.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1295.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1236.0) |
| KB2267602 | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1059.0) |

|  |                             |   |
|--|-----------------------------|---|
| KB2267602  | LiveStream<br>StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus -<br>(Version 1.353.1012.0) |
| <b>MedPresence™ Tower v10.3</b>  |                             |   |
| KB5008207  | MedPresence<br>Tower        | 2021-12 Cumulative Update for Windows 10 Version 1607 for<br>x64-based Systems            |
| <b>PHP Updates</b>   |                             |   |
| 7.1.33 and 7.4.16 – shipping versions.<br>7.3.33 is available upon request for v10.2.2 VaultStreams only.<br>7.4.26 is available upon request for v10.4 VaultStreams only. |                             |   |

For further questions concerning this notice, please call Technical Support at 1-800-848-9024