



## Microsoft® Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite HD and ES4K, nCare MX/RX, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

**Products:** EasySuite HD, ES4K and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

**Date:** May 10, 2022

### **Critical Security Bulletins (Obtained from Microsoft):**

**NOTE:** Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

### **Approved Updates:**

May 2022 Updates		
KB Number	Platform	Title
<b>VaultStream v10.x Updates – Windows® Server 2012</b>		
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.362.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.375.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.386.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.423.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.448.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.471.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.483.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.509.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.519.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.564.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.577.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.606.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.629.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.663.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.688.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.704.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.741.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.802.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.824.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.852.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.867.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.886.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.914.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.932.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.950.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.989.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1026.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1031.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1071.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1080.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1109.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1117.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1167.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1200.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1218.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1229.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1247.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1253.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1295.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1340.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1354.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1384.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1411.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1436.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1467.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1488.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1530.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1534.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1577.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1579.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1611.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1613.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1625.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1647.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1657.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1699.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1735.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1743.0)
KB5014011	VaultStream 2012	2022-05 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.101
KB5013872	VaultStream 2012	2022-05 Security and Quality Rollup for .NET Framework 3.5, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB5014025	VaultStream 2012	2022-05 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.363.1784.0)

<b>VaultStream v10.x Updates – Windows® Server 2016</b>		
KB5013952	VaultStream 2016	2022-05 Cumulative Update for Windows Server 2016 for x64-based Systems
KB5014026	VaultStream 2016	2022-05 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB5013625	VaultStream 2016	2022-05 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.101
<b>VaultStream v10.4 Updates – Windows® Server 2019</b>		
KB5013868	VaultStream 2019	2022-05 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64
KB5013941	VaultStream 2019	2022-05 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
KB5012796	VaultStream 2019	2022-04 Cumulative Update Preview for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64
KB5012636	VaultStream 2019	2022-04 Cumulative Update Preview for Windows Server 2019 (1809) for x64-based Systems
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1568.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1679.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1699.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1730.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1784.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1825.0)
KB890830	VaultStream 2019	Windows Malicious Software Removal Tool x64 - v5.101
<b>nCare MX v10.2.3</b>		
KB5013625	nCare MX	2022-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5013952	nCare MX	2022-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5014026	nCare MX	2022-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>nCare MX v10.4</b>		
KB5013625	nCare MX	2022-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5013952	nCare MX	2022-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5014026	nCare MX	2022-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.4 ES4K Controller (a.k.a. BlizCon)</b>		
KB5013625	ES4K Controller	2022-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5013952	ES4K Controller	2022-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems

KB5014026	ES4K Controller	2022-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.4   ES4K Video Manager (a.k.a. nCare RX2)</b>		
KB5013625	ES4K Video Manager	2022-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5013952	ES4K Video Manager	2022-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5014026	ES4K Video Manager	2022-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5011570	ES4K Video Manager	2022-03 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.4 LiveStream StatusBoard 2</b>		
KB5013868	LiveStream StatusBoard 2	2022-05 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64
KB5013941	LiveStream StatusBoard 2	2022-05 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1704.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1647.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1543.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1498.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1436.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1379.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1325.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1265.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1221.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1167.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1092.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1054.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.1004.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.939.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.886.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.851.0)



KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.817.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.716.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.663.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.602.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.544.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.505.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.453.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.386.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.336.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.363.663.0)
<b>MedPresence Tower v10.4</b>		
KB5013625	MedPresence Tower	2022-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5013952	MedPresence Tower	2022-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5014026	MedPresence Tower	2022-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>PHP Updates</b>		
<p>7.4.16, 7.1.33 and 5.6.40 – shipping versions.                      V10.2.2 was Tested with PHP version 7.3.33                      V10.4 was Tested with PHP version 7.4.29                      7.3.33 is available upon request for v10.2.2 VaultStreams only.                      7.4.29 is available upon request for v10.4 VaultStreams only.</p>		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024